

MK-9T SPA Digital Hydrotherapeutic Chamber

During the long human history, all therapeutic methods for diseases have been contributed to the relationship of water and sweat. Going for bath in hot spring or mineral spring for health has been found in the early age of human history. HINDU JAYURVEDA, one of the known earliest medical works, has recorded that steam bath can promote health. Cythians and Greek used steam bath at the same stage, which has been recorded in other famous works, such as HERODOTOS' TALES. We also know that Teutons always pour hot water on baked stone; Jews established bath room usually near to their houses; up to now, doctors of African tribes still recommend steam bath for treating some diseases.

The custom of sauna bath in Finland can be traced to as early as thousands years ago or even earlier, during which, the main features of Finland sauna bath has been kept. Sauna bath has been recognized as the national quintessence of Finland. Nowadays, 90% Finlanders conduct at least once sauna bath weekly. There are approximately 1.7 million sauna rooms in Finland, nearly one sauna room every three people. Although the art of pouring water on baked stone when sauna bath is not invented by Finlanders, then maintained and developed sauna. Finland has become the forerunner of modern sauna. Sauna is recognized as the heritage of Finlanders in Baltic Sea area, which is proved by the fact that the word sauna can be found in almost all languages. Yidi 2001 Finland Sauna Series is aimed to bring the culture of Finland sauna into families, as well as the healthy sauna into your healthy family. You may enjoy sauna and healthy life everyday without going out.

Configurations and functions

- 1 Sauna steam
- 2 Steam bath
- 3 Weiqi hydrotherapy (12 water power massage nozzle)
- 4 Nozzle shower bath
- 5 SPA medical herb (formulated) bath
- 6 Algae bath
- 7 feet liquid crystal TV set with closed circuit television
- 8 Red, blue, green light spectrum therapy
- 9 Dynamic fluorescence display

10 Powerful massage bath

11 Music therapy

12 Slim

13 Relaxation

14 Water bed

Suggestions:

1. Preliminary cleaning—dry steam — Cold shower bath — wet steam — Cold shower bath (warm)

2. Wet steam — Cold shower bath — dry steam — Cold shower bath(warm)

Scope of health care: fitness, acupuncture, massage, fatigue removal, body shaping, skin care, detoxification, beauty care, with physical therapy to stomach convulsion, rheum, arthritis, rheumatism, and seborrheic dermatitis

Operation:

Press On/Off switch to turn on the steam engine system. After rolling twice, the radio frequency and the real temperature of the body may be displayed. Press any function button to perform function operation.

Steaming function

1. Temperature configuration: Press the Temperature/time selector on the control box or the remote control, and the initial temperature value will be displayed (60 °C). When the temperature configuration indicator is on, press "temp, time adjust + or -" to adjust your desired steam bath temperature (0-68 °C). The actual temperature will be displayed 2 seconds after stop pressing the button.
2. Time configuration: Press Temperature/time selector on the control box or remote control, and the configured time will be displayed (30 minutes for default). Press "+" or "-" to adjust it to your desired value, ranging from 0 to 99.

Function operation:

- (1) Press the corresponding buttons on the control panel to turn on the lamp or fan. Press them again to shut down them. The working conditions are displayed on the display er (No signal displayed of

the lamp is shut down).

(2) Radio CD

1. Press the CD/Radio selector once to turn on the radio system. The radio status and the condition and frequency of last use are displayed. Press "MEM" and "TUN" buttons to automatically search broadcasting frequency from 87.5 to 108 MHz. Press "MEM"+ digit buttons to store the available broadcasting frequency. Press "TUN+" or TUN "-" button to fine tune the frequency. A total of 40 frequencies can be stored, Press "Call" + digit button to recall the stored frequency. If the radio number is more than 9, a "0" button should be added, i.e., press "CALL+0+4" for the No. 14 radio broadcasting.
2. Press the CD/Radio selector again to turn on CD playing system. (This machine is of CD plug, but the CD player and connector is not included). Press the button once more to shut down both radio and CD function.

(3) Ozone

Press O₃ button to start up the ozone function. Press it again to shut down it. The ozone function may be automatically shut down it 5 minutes after starting up.

Note: The ozone function is automatically shut down when the streaming and water heating function is started up. After safety consideration, it is designed to shut down ozone function when the streaming and water heating function is started up.

(4) Alarming

In case of accidents, press the alarming button to emit alarming sound. Except the lamp and fan, all functions are automatically shut down. In this condition, except the alarming button, all buttons are invalid. To restore to normal working condition, the alarming function should be shut down. The steam temperature and time, water heating temperature and radio frequency may be restored, while other functions may not be available until the machine is reset.

(5) Shut down

Press "On/Off" switch to shut down all functions. Open the steam water discharge valve to automatically discharge water (about 3 minutes). After discharging water, the machine can automatically shut down all electric supply to make it safe and dependable.

Note: Draw out the main power switch if does not use it for long.

Technical indicators

Parameters		Values
Rated voltage		AC220V \pm 10% (50-60Hz)
Rated power of main machine		3kW/4.5kW(+5%,-10%)
Power of distributor box		80W
Rated current		14A/20A/27A
Dimention	Main machine	225×90×250mm
	Distributor box	260×200×90mm
	Control box	190×135×40mm
Voltage tolerance		1250V/no striking through for 1 minute
Insulative resistance		\geq 7M Ω
Illumination lamp		AC12V/10~20W
Exhaust fan		DC12V/3W
Ozone jeneration		DC12V/5W
AC fuse		0.5A,250V
Reception of frequency range		(FM)88-108MHZ
Volume		(0-50db)adjustable
Max output power		30W
Speaker		Moistureproof speaker
Hot water and steam adaptor		Initial state
Timer		Initial state minutes

Trouble shooting

Trouble shooting

Troubles	Causations	Solutions
No display on the control box	1. Electric supply failure or the line is not well connected.	1. Wait for electric supply, correctly connect line.
	2. The on-off or earth leakage switch is not turned on.	2. Turn on the switch
	3. Incompact connection of the control box	3. Reconnect
No sound	1. Loosened connecting line	1. Reconnect the line
	2. The volume is set too low.	2. Reset the volume
	3. The speaker is damaged.	3. Replace the speaker.
Bad radio signal	1. The frequency is not properly modulated.	1. Remodulate frequency signal.
	2. The antenna is not properly hung.	2. Properly adjust antenna
Lamp is not on.	1. The connecting line or lamp holder is loosened.	1. Reconnect or reinstall.
	2. The bulb is damaged.	2. replace
The ventilator does not rotate.	1. Loosened connecting line	1. Reconnect the line
	2. blocked ventilator	2. Remove blocking mess or replace
	3. The ventilator is damaged.	3. Replace
No steam	1. No water supply or bended water inlet pipe	1. Wait for water supply or replace water inlet pipe.

	2. Obstructed or damaged water inlet electromagnetic valve	2. clear up or replace.
remote control failure	Out of battery or the button is damaged	replace battery or send back to factory.
And the machine does not work with the electricity leakage indicator on	1. 1.5A AC fuse is open.	1. Replace the 1.5A fuse.
	2. The transformer is overheated and the over loading protector is damaged.	2. Replace the transformer.
Can not receive or dial telephone	The telephone wire in not connected or contact failure	Reconnect the telephone line.
no ozone output	1. The ozone output conductor is not well connected.	1. Reconnect the conductor.
	2. The steaming function is on use.	2. Shut off the steaming function.
	3. The parameters of the ozone generator is not appropriate or it is damaged.	3. Replace the ozone generator with a new one.

If troubles remain, please contact with our company for maintenance. Do not disassemble the main machine and the controller by yourself. We are free of responsibility if the machine is disassembled and damaged.

MIRACLE CO.,LTD

Address: Room B601 Hengsheng Building B , No. 205 Huanshi Zhong Road Guangzhou China /POST NO.:510010

TEL:86-20-86673107 FAX:86-20-86672750

Email: admin@miracle-china.com

Skype: miracle-china

MSN:china.miracle@hotmail.com QQ:286372662

<http://www.miracle-china.cn>

www.miracle-china.cn