

Far Infrared Ray Energy Capsule

Product Introduction



Brief introduction

This is the perfect combination of colored light fragrance medical therapy with electronic technology ,traditional Chinese philosophy and the essence of western science and technology. Being in this magic cabin, your heart and soul will be just like flying in the enormous universe, while improving the metabolism by the resonance massage of fat cells with infrared rays; or even immersed in the forest full of anions and vitality, remolding your skin thoroughly.

Main curative effects:

A. thermal energy therapy:

The infrared light wave has effects on the subcutaneous fat tissue, burning the fat and removing it with sweat, within 20~30 minutes, 600 calories is consumed, equal to ten thousand -meter-long running. The temperature is up to 90°C, helpful to improve lymph circulation, remove toxins and unnecessary water; besides, it is effective to treat arthritis and rheumatism as well.

B. fragrance therapy:

Different kinds of essences of plants through the essence outlet are taken in to the internal organs, gaining physical and spiritual curative effects.

C. colored light therapy:

The colored light waves from particular light source can stimulate the metabolism of cells, remove skin diseases based on the brand new concept of optical therapy.

D. anion breeze therapy:

A large amount of anions are especially helpful to remold your skin and relax muscles.


E. ultraviolet sterilizing system:

Thoroughly cleansing the tiny pores, getting rid of the toxic heavy metals.

F music therapy

CD playing system makes you relaxed and refreshed physically and mentally.

Operating Instruction

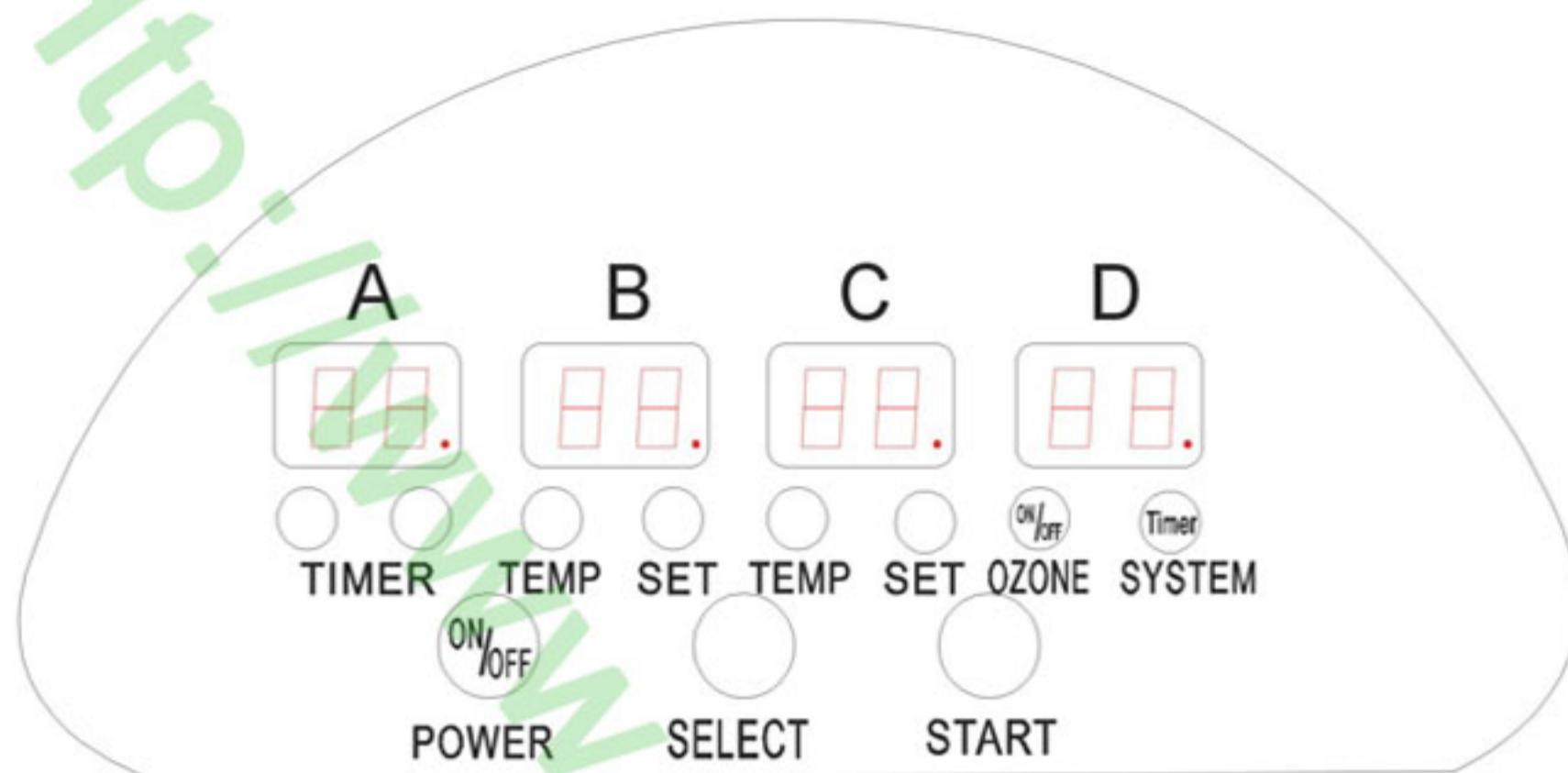

Control panel introduction

POWER: main power of the whole equipment

SELECT: the upper capsule and lower capsule work exchange press it , the upper capsule is worked.press again, the lower capsule is worked, press Three times, the upper and lower capsule is worked.

START: the system start working .

A: time display

B: the upper temperature display

C: the lower temperature display

D: ozone system time display


TIMER: the system time control

TEMP SET: temperature control

Timer : ozone system time control

Operation instructions

1. Press "POWER" button for system on. Press again for system off..
2. Press "SELECT" button for the upper capsule and lower capsule is worked.
3. Press "TEMPSET" button for set temperature, subject about 70 °C to 90 °C
4. Press "TIMER" button for set system time, subject about fifteen to thirty minutes.
5. Press "START" button for star system working .
6. Press "OZONE" button for ozone on, press again for system off.
7. Press "SYSTEM" button for set ozone time , subject around ten minutes.
8. Press "POWER" button for system off.

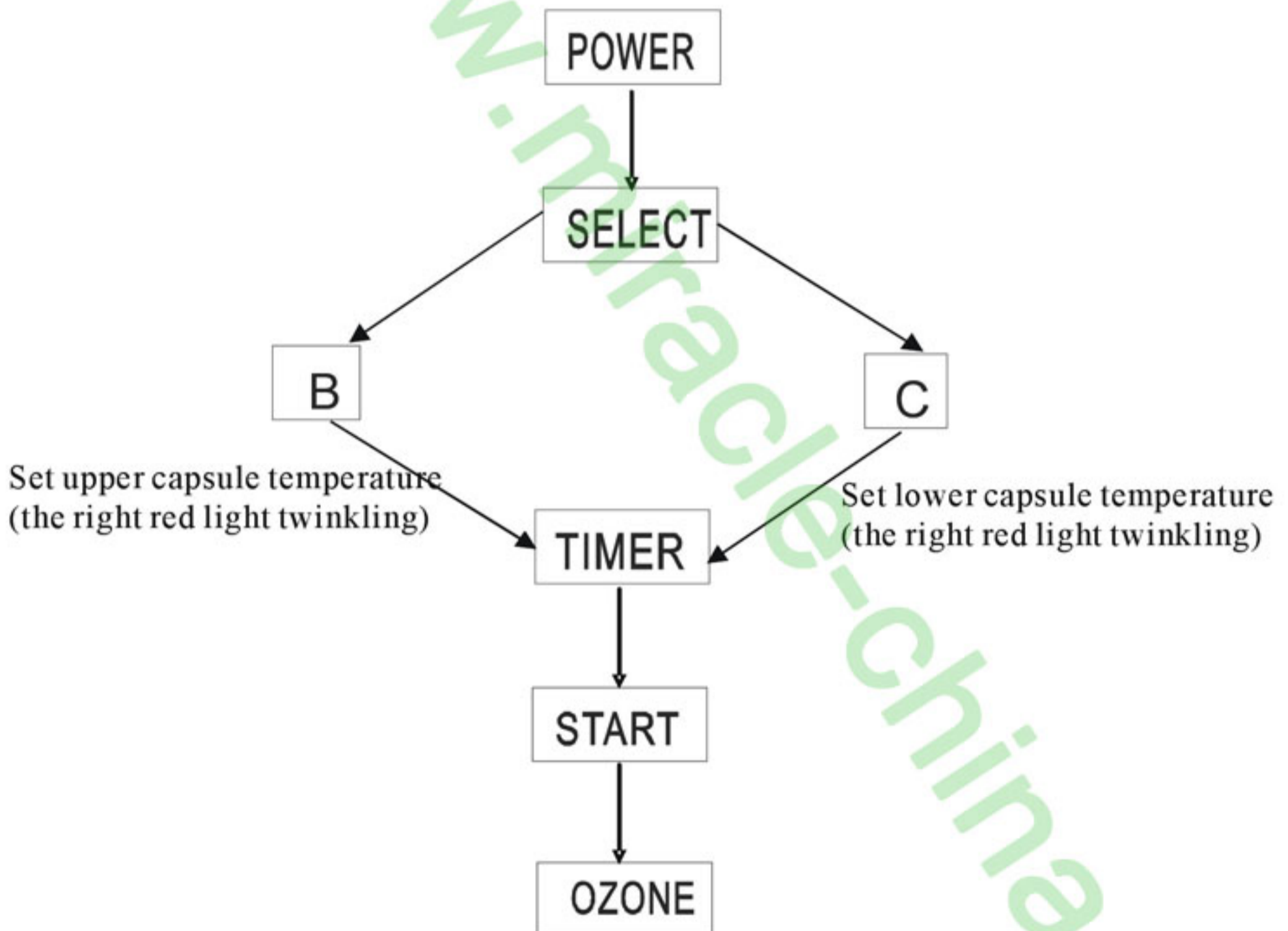

The function of wideband for infrared

It's a kind of electromagnetic wave, an intense radioactive ray with longer wavelength than the visible light on the spectrum. The near infrared can be used in food processing, vehicle painting, cooking, while the far infrared is particularly suitable for physiotherapy. It can be obtained only if light is accompanied with heat.

Why it is warm

Molecular movements can be changed to resonance vibration in the infrared, this vibration will generate frictional heat. It can penetrate 10mm into the epidermis in comparison with the normal heat stimulation 0.5mm, so the infrared makes people warm.

What the relationship between the far infrared and health

- A.. Improve blood circulation and metabolism.
- B . Relieve pain due to the heat on the nerve endings.

The physiological effect of the far infrared

1. Improve the physiological oxidation and reduction, remove the redundant fat, and keep fit.
2. Save your energy, not like the exercise does.
3. Heat preservation effect can maintain your body in a balanced nutrition.
4. Speed up perspiration and fat metabolism (your body should have enough water supply) as well as the removal of poison, heavy metal, waste, etc.
5. Stimulate physiological function, intensify the hormone and enzyme.
6. Build up your health, release fatigue.
7. Physiotherapy with muscular and neural ailments, relieve liver tension and pain.
8. Prevent the spread of tumors.
9. Release pain and diminish inflammation.
10. Regenerative function.

Dismantling instruction

1. Remove the wooden mattress, there is a rectangular cover, take off the ten screws.
2. In the middle there is the main engine. Plug off all connection with upper capsule carefully.(for convenient assembling, take notes of all positions).
3. Take off the springs on the side rods, dismantle the supporting rod .
4. Two people lift up the upper capsule, another person take off the screws on the hinge with a small screwdriver, pull out the hinge bolt, then the upper capsule separates with the lower capsule the dismantling is finished .

Rated voltage: 220V

Rate electriccurrent: 7A

Rated power: 2100w

NOTE

- It's a high-tech instrument, dismantling is not allowed.
 - Prevent from water, fire, humidity and corrosion.
 - Away from interference while being used.
 - When pressing the button, sharp things are not allowed.
 - For the sake of avoiding interference with computer chips.
- Don't start the infrared light and the rise-fall system at the same time , if it happens unplug for 5 seconds, then start again.
- If the power doesn't work, check the fuse first. Replace with the same type when the fuse is burned.
 - Standard electric current 25A, the plug is three-phase. Make sure to attach the earth wire.
 - Turn off the instrument and then unplug it.
 - Don't use detergent containing corrosive or oxidizing agent.
 - It's dangerous for people with heart disease , high-blood pressure or pregnant women.

Letter of guarantee

With this card service is free during a year

| | |
|--------------|-----------------------------|
| QC NO: | |
| MODEL | MK-123 |
| PERIOD | One year after purchasing |
| DATE OF SALE | month date year |
| BUYER | Name: Address: Phone: |
| DISTRIBUTOR | Name: Address: Phone: |

MIRACLE CO.,LTD

Address: Room B601 Hengsheng Building B , No. 205 Huanshi Zhong Road Guangzhou
China /POST NO.:510010

TEL:86-20-86673107 FAX:86-20-86672750

Email: admin@miracle-china.com Skype: miracle-china

MSN:china.miracle@hotmail.com

QQ:286372662

<http://www.miracle-china.cn>

www.miracle-china.cn